



DTX Studio™ Core

Release notes

Release date: 2025-02-14

Table of Contents

Version 4.1.4	4
Compatibility	4
New Features	4
Audit Logs	4
Enhancements	4
General	4
OP 3D Vision	5
DICOM	5
Workflow Efficiency	5
Most Important Bug fixes	5
Previous Releases	6
Version 4.0.6	7
Compatibility	7
Most Important Bug Fixes	7
Version 4.0.5	8
Compatibility	8
Most Important Bug Fixes	8
General	8
DTX API	8
Migration Control Center	8
i-CAT Capturing	8
Version 4.0.3	9
Compatibility	9
New Features	9
DICOM Module	9
DTX Studio Clinic Connections	9
Enhancements	10
Installation and Configuration	10
Duplicate PMS IDs	10
Logging	10
Most Important Bug Fixes	10
General	10
DTX Studio Core Capturing	10

Version 3.15	11
Compatibility	11
New Features	11
Central Data Storage	11
PMS Bulk Correction	11
Enhancements	12
Audit Logging	12
Enable DTX Studio Core Scan Center	12
Security	12
https Support	12
Performance Improvements	12
Most important bug fixes	12
General	12
Migrations	12
DTX Studio Core Capturing	13
Version 3.13	14
Compatibility	14
New Features	14
Migrations	14
Enhanced Snapshot	15
Templates and QuickPrescribe	15
PMS Service	15
Silent Install of DTX Studio Core	15
Usage Report	15
Enhancements	15
Https Protocol for Communication with DTX Studio Clinic	15
i-CAT™ and OP 3D Vision™ Integration	16
Extended Compatibility	16
Most Important Bug Fixes	16
General	16
Migrations	16
DTX Studio Core Capturing	16

Version 4.1.4

This new version of DTX Studio Core introduces some robustness and stability improvements. Moreover, the new Dexis Capture application is supported.

Note

All fixes and improvements are incremental to DTX Studio Core 4.0.6.

Compatibility

DTX Studio Core 4.1 is compatible with **DTX Studio Clinic 3.1.1 or higher**.

DTX Studio Core 4.1 needs **DTX Studio Driver 21.1 or higher**.

DTX Studio Core 4.1 needs **DTX Studio Database Migrations 3.11.1** or higher for performing migrations. Compatible plugins version 7 are required. (DTX Studio Core 4.1 will not connect to older DTX Studio Database Migration versions and plugins).

Important

Make sure, **all** computers in the clinic are upgraded from previous versions to the **same DTX Studio Clinic version**.

For 3D devices, please check firmware compatibility with DTX Studio Driver. A firmware update may be required. When updating firmware, OP 3D devices may require recalibration by a service technician on site. For device and firmware compatibility information, please refer to DTX Studio Driver technical bulletins available at <https://tech.dexis.com>.

New Features

Audit Logs

Actions performed in the DTX Studio Core web portal are now added to a dedicated audit log. Audit logs can be enabled or disabled in the configuration wizard.

Enhancements

General

- CliniView is no longer supported as an integrated frontend. When upgrading to this new version of DTX Studio Core a notification message is shown.
- The DTX Studio Core scan center is no longer available.
- The folder structure in which actual media (data) are stored in DTX Studio Core was reworked to increase robustness.
- A banner is now shown on the DTX Studio Core web pages when an optional software update is available.
- On the DTX Studio Core login page, an action is now available that redirects to a DTX Studio Go web page that allows to request a new login password.
- Users can change a configured SCU storage destination.
- A configured modality worklist SCU can be deleted or replaced.

OP 3D Vision

The communication framework between DTX Studio Core and the OP 3D Vision (i-CAT) was updated and improved to allow faster data transfer between the acquisition device and DTX Studio Core. Moreover, thanks to the new framework, DTX Studio Core can directly interact with the acquisition workstation which eliminates the need of an additional data storage PC (PACS node).

DICOM

Merge, the internal third-party DICOM library, has been replaced by the dcmtk library.

Workflow Efficiency

- Scan requests which are overdue for a number of days, will be closed automatically.
- A new setting was introduced that allows to automatically accept any pending DTX Studio Clinic connection request.

Most Important Bug fixes

- Fixed issue where the RDSR is incorrectly stored under same series as media.
- Fixed issue where a reassigned image is deleted after deleting the original patient record.
- Fixed issue where the wrong area is used when creating RDSR for CBCT.

Previous Releases

Version 4.0.6

This version includes a bugfix related to cep/pan image quality.

Note

All fixes and improvements are incremental to DTX Studio Core 4.0.5.

Compatibility

DTX Studio Core 4.0 is compatible with **DTX Studio Clinic 3.1 or higher**.

DTX Studio Core 4.0 needs **DTX Studio Driver 21.1 or higher**.

DTX Studio Core 4.0 needs **DTX Studio Database Migrations 3.11.1** or higher for performing migrations. Compatible plugins version 7 are required. (DTX Studio Core 4.0 will not connect to older DTX Studio Database Migration versions and plugins).

Important

Make sure, **all** computers in the clinic are upgraded from previous versions to the **same DTX Studio Clinic version**.

For 3D devices, please check firmware compatibility with DTX Studio Driver. A firmware update may be required. When updating firmware, OP 3D devices may require recalibration by a service technician on site. For device and firmware compatibility information, please refer to DTX Studio Driver technical bulletins available at <https://tech.dexis.com>.

Most Important Bug Fixes

Fixed issue where OP 3D Pro cep/pan images received by PACS had reduced image quality.

Version 4.0.5

This version mainly includes important bugfixes related to MCC, image acquisition through i-CAT devices, certificate handling and the DTX Studio Core application in general.

Note

All fixes and improvements are incremental to DTX Studio Core 4.0.3.

Compatibility

DTX Studio Core 4.0 is compatible with **DTX Studio Clinic 3.1 or higher**.

DTX Studio Core 4.0 needs **DTX Studio Driver 21.1 or higher**.

DTX Studio Core 4.0 needs **DTX Studio Database Migrations 3.11.1** or higher for performing migrations. Compatible plugins version 7 are required. (DTX Studio Core 4.0 will not connect to older DTX Studio Database Migration versions and plugins).

Important

Make sure, **all** computers in the clinic are upgraded from previous versions to the **same DTX Studio Clinic version**.

For 3D devices, please check firmware compatibility with DTX Studio Driver. A firmware update may be required. When updating firmware, OP 3D devices may require recalibration by a service technician on site. For device and firmware compatibility information, please refer to DTX Studio Driver technical bulletins available at <https://tech.dexis.com>.

Most Important Bug Fixes

General

- Fixed issue where DTX Studio Core did not start after reboot or upgrade (specific cases).
- Fixed issue where DTX Studio Core failed to start when a revoked application has a patient lock.
- Fixed issue where the acquisition time was not as expected for migrated data when DTX Studio Core in different time zone.

DTX API

- Fixed issue where the certificate download endpoint required authentication while it should not.

Migration Control Center

- Fixed issue where incremental migration was not working as expected.
- Fixed issue where the migration from CLINIVIEW 10 reported unexpected failures.
- Fixed issue where migration failed when migrating from specific very large legacy databases.

i-CAT Capturing

- Fixed issue where i-CAT scans made through Quick prescribes were not added to the patient file.
- Fixed issue where images captured with i-CAT were not available on DTX Studio Core when the patient name contains special characters.

Version 4.0.3

This version mainly introduces support for a new DICOM communication module, an enhanced connection process between DTX Studio Clinic and DTX Studio Core as well as a number of stability improvements.

Note

All fixes and improvements are incremental to DTX Studio Core 3.15.

Compatibility

DTX Studio Core 4.0 is compatible with **DTX Studio Clinic 3.1 or higher**.

DTX Studio Core 4.0 needs **DTX Studio Driver 21.1 or higher**.

DTX Studio Core 4.0 needs **DTX Studio Database Migrations 3.11.1** or higher for performing migrations. Compatible plugins version 7 are required. (DTX Studio Core 4.0 will not connect to older DTX Studio Database Migration versions and plugins).

Important

Make sure, **all** computers in the clinic are upgraded from previous versions to the **same DTX Studio Clinic version**.

For 3D devices, please check firmware compatibility with DTX Studio Driver (formerly KaVo Driver). A firmware update may be required. When updating firmware, OP 3D devices may require recalibration by a service technician on site. For device and firmware compatibility information, please refer to DTX Studio Driver technical bulletins available at <https://tech.dexis.com>.

New Features

DICOM Module

A new DICOM communication module is introduced. The DICOM module allows to forward acquired media to a configured PACS system (DICOM Storage SCU), to receive DICOM worklist items (MWL) from a configured PACS or RIS system and finally to create and send DICOM radiation dose reports (RDSR). When DICOM MWL items are received, these items are automatically converted to DTX Studio Core scan requests. Radiation dose reports (RDSR) can be enabled for a configured DICOM storage destination. Dedicated configuration pages and settings are available in the DTX Studio Core management web portal. Finally, dedicated audit logs are available in this new DICOM module to offer full traceability on sending and receiving data.

DTX Studio Clinic Connections

The process to connect a DTX Studio Clinic installation to DTX Studio Core was further improved. Connection requests are now listed in the DTX Studio Core web portal, where requests can be accepted or rejected by the system administrator.

The new connection process no longer requires DTX Studio Clinic users to enter the DTX Studio Core username and password.

Moreover, the new management web portal page offers a quick overview of linked DTX Studio Clinic installations. For each installation the current software version and the 'last active date' is listed.

Note

To use all features of the new connection workflow, DTX Studio Clinic 4.3.10 or higher is required.

Enhancements

Installation and Configuration

- The default port numbers have been changed. Details on the used port numbers can be found in the DTX Studio Core 4.0 installation guide.
- An additional notification was added to the configuration wizard, to guide the user to the DTX Studio Core activation process.
- The DTX Studio Core root certificate is now automatically installed.
- DTX Studio Core can now be activated silently through the DSO API.

Duplicate PMS IDs

An additional constraint was added to DTX Studio Core to avoid duplicate PMS IDs in the DTX Studio Core database.

Logging

The logging of OP 3D (LX) reservation error info has been improved.

Most Important Bug Fixes

General

- Fixed issue where the activation of DTX Studio Core failed in a specific case (duplicate DTX Studio Go user).

DTX Studio Core Capturing

- Fixed issue where calibration images can no longer be downloaded.
- Fixed issue where unknown characters appeared in the device details page when using Scan eXam One.
- Fixed issue where the device connection ended unexpectedly, resulting in a non-cleared device (specific case).

Version 3.15

This release brings performance improvements, stability improvements and bug fixes for connections with devices and connected workstations with DTX Studio Clinic.

Note

All fixes and improvements are incremental to DTX Studio Core 3.13.

Compatibility

DTX Studio Core 3.15.3 is compatible with **DTX Studio Clinic 3.1 or higher**.

DTX Studio Core 3.15.3 needs **DTX Studio Driver 21.1 or higher**.

DTX Studio Core 3.15.3 needs **DTX Studio Database Migrations 3.11.1** or higher for performing migrations. Compatible plugins version 7 are required. (DTX Studio Core 3.15 will not connect to older DTX Studio Database Migration versions and plugins).

Important

Make sure, **all** computers in the clinic are upgraded from previous versions to the **same DTX Studio Clinic version**.

For 3D devices, please check firmware compatibility with DTX Studio Driver (formerly KaVo Driver). A firmware update may be required. When updating firmware, OP 3D devices may require recalibration by a service technician on site. For device and firmware compatibility information, please refer to DTX Studio Driver technical bulletins available at <https://tech.dexis.com>.

New Features

Central Data Storage

Users can now upload to DTX Studio Core data files that should be distributed within the practice to connected DTX Studio Clinic clients. This should facilitate the process to share and distribute certain configuration files throughout the practice.

Following data file types are now supported:

- DTX Studio Clinic shared setting files
- DTX Studio Clinic installers
- DTX Studio Clinic Implant Library packages

PMS Bulk Correction

DTX Studio Core now allows to export an overview (csv file) of all patient records in DTX Studio Core.

The system also allows to import an altered csv file to perform bulk corrections on the patient records (example correct all internal PMS ID's).

Multiple checks are performed automatically on the csv file to prevent unintentional changes.

Enhancements

Audit Logging

Audit logs are created for login attempts to DTX Studio Core (and linked components).

Enable DTX Studio Core Scan Center

The DTX Studio Core Scan Center can now be enabled in the configuration wizard. By default the DTX Studio Core scan center is disabled.

Security

Security improvement for scan center login (cross-origin error).

https Support

DTX Studio Core has now full https support on all available DTX Studio Core services.

Performance Improvements

- Multiple technical performance improvements on DTX Studio Core API.
- DTX Studio Core can now support up to 40 active DTX Studio Clinic clients.

Most important bug fixes

General

- Fixed issue where the summary report incorrectly counts patients and media in recycle bin.
- Fixed issue where the patient name is printed in DTX Studio Core log file when patient name is changed in Cliniview.
- Fixed issue where an incorrect error message was shown when the configuration wizard could not be started.
- Fixed issue where the scheduled time for a DTX Studio Core snapshot not correctly set.

Migrations

- Fixed issue where DTX Studio Core hangs during Mcc upgrade (h2) with a very large migration database.
- Fixed issue where a patient was not merged during migration if same PMS ID already exists in DTX Studio Core
- Fixed issue where media of second patient was not migrated automatically when merging patients.

DTX Studio Core Capturing

- Fixed issue where an unassigned image could not be assigned nor discarded.
- Fixed issue where the acquisition time should be stored in local time when capturing 3D data through DTX Studio Core.

Version 3.13

This release brings performance improvements, stability improvements and bug fixes for connections with devices and connected workstations with DTX Studio Clinic.

Note

All fixes and improvements are incremental to DTX Studio Core 3.10.5.

Compatibility

DTX Studio Core 3.13 is compatible with **DTX Studio Clinic 1.9.2 or higher**.

DTX Studio Core 3.13 needs **DTX Studio Driver 21.1 or higher**.

DTX Studio Core 3.13 needs **DTX Studio Database Migrations 3.11** or higher for performing migrations. Compatible plugins version 7 are required. (DTX Studio Core 3.13 will not connect to older DTX Studio Database Migration versions and plugins).

Important

Previous image capturing workflow for i-CAT™ or OP 3D Vision is replaced with a new simplified workflow without the use of the Modality Worklist (MWL). Note that the previous workflow (using MWL) is no longer supported when upgrading to version 3.13 or higher.

The minimum supported version of SmartScan STUDIO is 2.12 or higher.

Make sure, **all** computers in the clinic are upgraded from previous versions to the **same DTX Studio Clinic version**.

For 3D devices, please check firmware compatibility with DTX Studio Driver (formerly KaVo Driver). A firmware update may be required. When updating firmware, OP 3D devices may require recalibration by a service technician on site. For device and firmware compatibility information, please refer to DTX Studio Driver technical bulletins available at <https://tech.dexis.com>.

New Features

Migrations

The Migration Services: DTX Studio Core supports sequential migrations of different legacy and other supported third-party databases. Technology used for migrations has been changed for better support of large database migrations. New and existing one-time and side-by-side migration setups now allow for faster operations with migrated data.

The migration framework was further extended and improved to smoothen the overall migration process. The Migration Control Center now lists more detailed information on the configured migration: number of patients, number of patients with media, patients with a conflict, a list of the images that failed during migration.

Conflicts that occurred during migration are now listed and can optionally be resolved in Migration Control Center (MCC). Two patient records can now be merged. The “Source patient” record can be merged into the “Target patient” record. Media are merged, available custom media are rewritten. The source patient record is deleted.

Enhanced Snapshot

A health check is now performed before a snapshot is taken, to prevent corrupted snapshots. Health notifications are added in case the DTX Studio Core database snapshot has failed or if no snapshot is scheduled. Furthermore, only one snapshot job can run at a time (to prevent errors and DTX Studio Core overloads).¹

Templates and QuickPrescribe

When defining a new QuickPrescribe scan protocols on DTX Studio Core, a preferred capturing template can be set, and users can specify whether a diagnostic region should be selected for the linked scan requests.

QuickPrescribe and templates can be imported by copying a template or QuickPrescribe configuration file to the correct DTX Studio Core folder.

PMS Service

A new PMS Service was implemented in DTX Studio Core. This service allows to create or update patient records stored in DTX Studio Core from a third-party application (for example a PMS system). Moreover, for each patient record, information about the stored 2D and 3D media can be retrieved. Third-party systems can create Scan Requests by using this new PMS service. A special configuration page is available to enable the PMS Service and configure some service details. The PMS service also allows to create a DTX Studio Core database snapshot.

Silent Install of DTX Studio Core

Both the DTX Studio Core installer and the Configuration wizard can be run silently, via a command line with parameters. The parameters for the Configuration wizard are Auto-accept EULA and Load a Core Configuration parameters file (optional).

Usage Report

Customers can now have access to a Usage Report. This report provides an overview of the daily usage of DTX Studio Core (number of patients, number of media, linked devices and other).

The Usage Report feature is by default OFF and can be activated upon request.

Enhancements

Https Protocol for Communication with DTX Studio Clinic

The more secure encrypted https protocol is now the default communication option. Users need to explicitly enable http communication if this is required.

¹ For default PostgreSQL database type only

i-CAT™ and OP 3D Vision™ Integration

The integration between DTX Studio Core and i-CAT or OP 3D Vision was further improved. Scan requests or capture sessions are now directly communicated through the native i-CAT or OP 3D Vision integration API (Atlas Web Service) instead of using the Modality Worklist service.

Extended Compatibility

DTX Studio Core 3.13 is additionally supported on Windows 11 and Windows Server 2022.

For technical requirements refer to the latest computer guidelines.

Most Important Bug Fixes

General

- Fixed issue where upgrading DTX Studio Core from version 3.4.7 to version above 3.9 failed.
- Fixed issue where DTX Studio Core services could not start due to empty or corrupted configuration files.
- Fixed issue where upgrade from DTX Studio Core 3.10 failed.
- Fixed issue where the 3Shape device status was not updated if the 3Shape Dental Desktop server went offline or online.
- Fixed issue where notification message to DTX Studio Clinic about unassigned images indicated a wrong date (day).
- Fixed issue where some patient records could not be opened in DTX Studio Clinic for installs with history of using DTX Studio Core version 3.7.

Migrations

- Fixed issue where a table in the MCC could not be sorted based on Message.
- Fixed issue where the login password in the MCC was not updated when the password was changed in DTX Studio Go.
- Fixed issue where the migration of a very large database would stall during the media metadata import phase.

DTX Studio Core Capturing

- Fixed issue where starting the OP 2D calibration sometimes failed.



Nobel Biocare AB
Box 5190, 402 26
Västra Hamngatan 1,
411 17 Göteborg,
Sweden

www.nobelbiocare.com

Distributed in Australia by:

Nobel Biocare Australia Pty Ltd
Level 4/7 Eden Park Drive
Macquarie Park, NSW 2114
Australia

Phone: +61 1800 804 597

Distributed in New Zealand by:

Nobel Biocare New Zealand Ltd
33 Spartan Road
Takanini, Auckland, 2105
New Zealand

Phone: +64 0800 441 657

Distributed in Turkey by:

EOT Dental
Sağlık Ürünleri ve Dış Ticaret A.Ş
Nişpetiye Mah. Aytar Cad.
Metro İş Merkezi No: 10/7
Beşiktaş İSTANBUL
Phone: +90 2123614901